


Whatever You Use, However You Use Test for Fentanyl!

Testing Powder (Heroin)

1. First things first, load your shot just as you usually do!
2. Next, add about 1/2-2/3 of a water vial to your **empty baggie** or the **cooker** you just used - we are testing rinse water!
3. Dip the end of the test strip (with wavy lines) into the water mixture and hold it for a count of **15 seconds**.
4. Lay the test strip flat on a sterile surface or on the top of your cooker for 1-2 minutes.

One line is POSITIVE for Fentanyl. Two lines is NEGATIVE for Fentanyl.

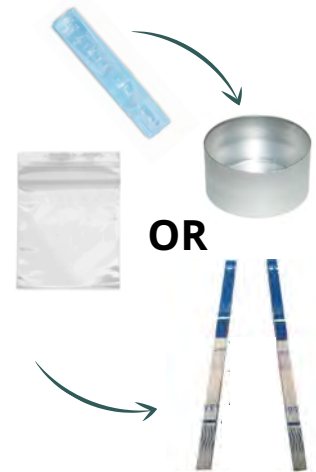

Testing Meth

Testing meth looks a little different! The composition of meth (or MDMA) can trigger a positive response if too high of a concentration is used.

1. Load your shot first!
2. Now just using the **residue from an empty baggie or cooker**, mix with **1/2 cup of water**.
3. Same as before, dip the end of the strip in the water mixture for 15 seconds.
4. Lay on a flat, sterile surface or across the cooker for 1-2 minutes.

One line is POSITIVE for Fentanyl. Two lines if NEGATIVE for Fentanyl.



Testing Cocaine, Ketamine, Pills, Non-Injectables

Once again we only need a little bit of product to test. **If testing pills**, break off a tiny piece OR cut in half and rub the two halves together to get a few granules - remember that Fentanyl in pressed pills will be unevenly distributed so the best method is to crush the whole pill and pull from that. **If testing powder**, remove a small amount from your supply. **We want about 6mg of product to test, this looks like a match head amount of powder**

1. Mix the collected powder with **1 ounce of water** - half a shot. If using less than 6mg of powder, adjust water accordingly.
2. Dip the end of the Fentanyl test strip into the water mixture, holding onto the blue end. Hold it in the water for **15 seconds**.
3. Lay the strip flat on a sterile surface for 1-2 minutes.

One line is POSITIVE for Fentanyl. Two lines is NEGATIVE for Fentanyl.

Be aware there is always a margin for error with Fentanyl Test Strips. Whether your supply tests positive or negative, go slow, have a buddy near by and have Naloxone on hand just in case.



Comments, Questions, Concerns? Get in touch!

HOLLER HARM REDUCTION

828-290-9066

HELLO@HOLLERHARMREDUCTION.ORG

Holler Harm Reduction is a community based Harm Reduction program, 501(c)3 non-profit operating in Madison County, NC.